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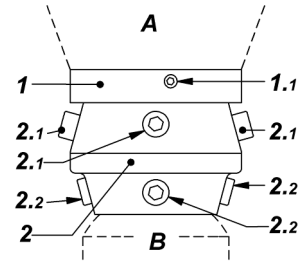
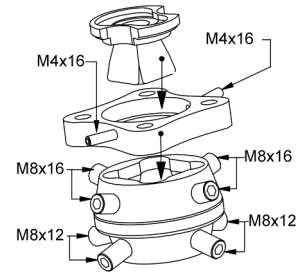
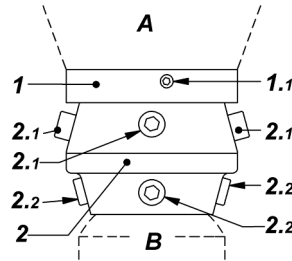
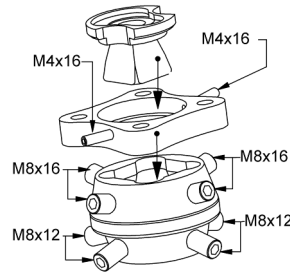
1K17099-0212

ASSEMBLING

Assemble the different parts of the 1K170 TAC module as shown in the diagram opposite.

CONNECTIONS

The 1K170 TAC module can be assembled :

- at its upper plate **1** to any component **A** with European 4-hole pattern (4 M6 tapped holes, centre distance 36 mm : e.g. 1K01, 1K90, ...)
- at its lower part **2** to any component **B** providing the male pyramidal connection (e.g. 1K160, 1K163, 1K90, ...)

CAUTION : *The upper plate 1 must necessarily be connected to the upper part of the prosthesis, and the lower part 2 to the lower part of the prosthesis.*

ADJUSTMENTS

- Shift (plate **1** / lower part **2**) is adjusted with screws **2.1** (range of adjustment is ± 7.5 mm in the frontal plane and ± 7.5 mm in the sagittal plane)
- Rotation (plate **1** / lower part **2**) is adjusted as follows : slightly loosen screws **2.1**, then use screws **1.1** to achieve the desired adjustment. Re-tighten screws **2.1**, then screws **1.1**
- Tilt is adjusted with screws **2.2** until the desired adjustment is achieved (range of adjustment is ± 9 mm in the frontal plane and ± 9 mm in the sagittal plane)

SCREWS TIGHTENING

- Screws **2.1** and **2.2** : recommended tightening torque : 15 Nm
- Screws **1.1** : recommended tightening torque : 4 Nm

We highly recommend to apply XC047 Proteor adhesive to screws **1.1**, **2.1** and **2.2** before tightening them.

CAUTION

Never leave one of the **2.1** or **2.2** screws in place if its head has penetrated more than one millimetre into the tapped hole. If this happens, replace the screw with one of the longer ones provided. Conversely, if a screw seems to jut out too much, replace it with one of the shorter ones provided.

USE

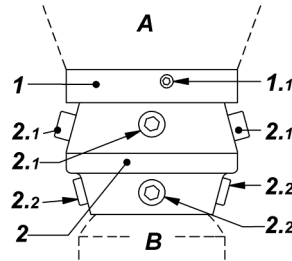
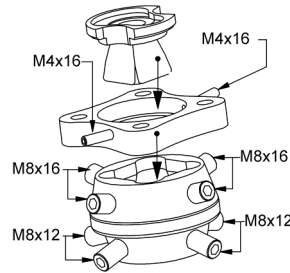
This module is appropriate for patients up to 100 kg, carried load included.

RECYCLING

Parts **1** and **2** are made of stainless steel, screws **1.1**, **2.1**, **2.2** and the internal pyramid in steel. Each of these items should be recycled according to the law in force.

6, rue DE LA REDOUTE - Z.I. SAINT-APOLLINAIRE
B.P. 37833 - 21078 DIJON CEDEX - FRANCE
PHONE : +33 3 80 78 42 10 - FAX : +33 3 80 78 42 15
ht.orthopedie@proteor.com - www.proteor.com

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