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INSTRUCTIONS : Seat with movable back 9Z009, 9Z017 and 9Z018

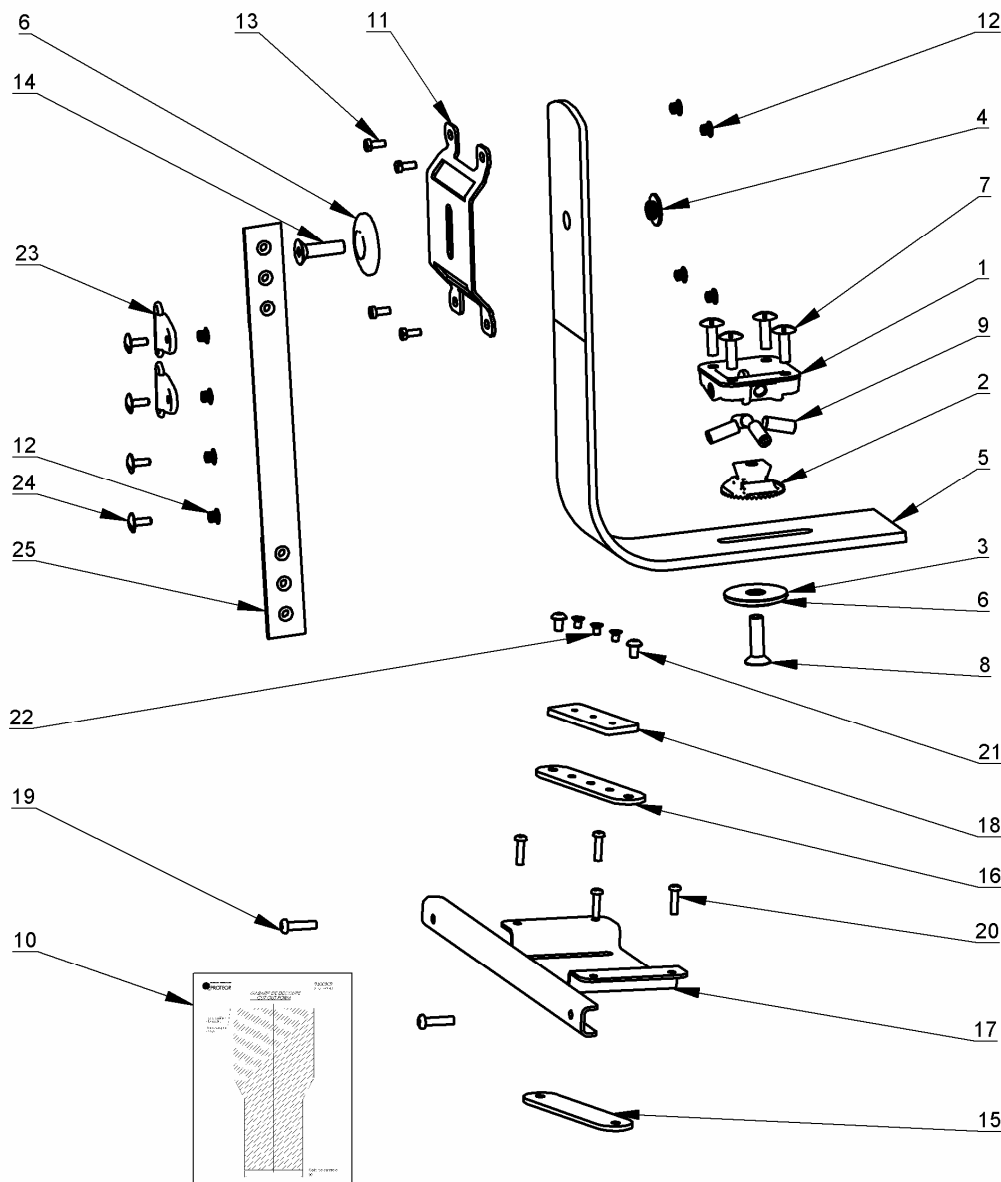
The seat with movable back is indicated for any pathology requiring a stabilization of the sitting position and the possibility of moving the trunk with regard to the pelvis, in an oriented way or not, temporarily or permanently.

The locking kit (ref. PROTEOR 9Z010 and 9Z021 for the reinforced model) links the back to the seat if necessary.


An adjustment allows to change the mast stiffness either globally, or on the right or the left side.

We recommend using this seat-brace with moving back for postural scheme development under supervision.

Please refer to the table of the following page to select the kit reference according to the patient weight and size.

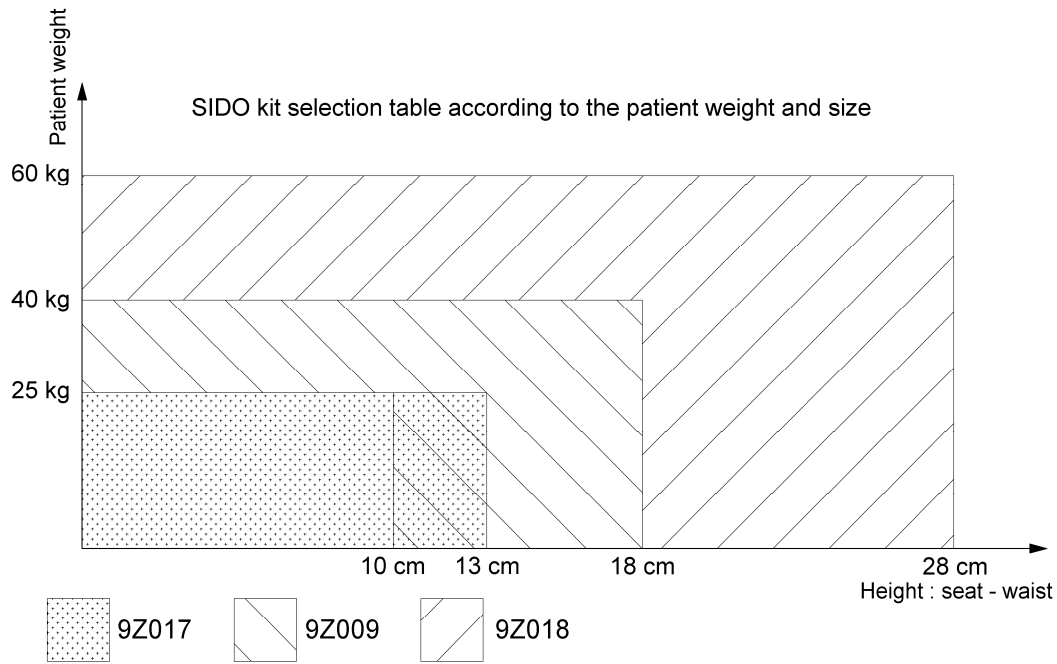For detailed information about kit mounting, please see technical manual MAN0026.

The kit must be installed on a shell after trial fitting, when the plumb lines and the cut-outs have been traced and when the base dimensions are known.

Minimum shell thickness : 6 mm.

When you cut out the shell, leave at least a 20 mm space between the valves (to prevent pinching risks).

If necessary use the wedge kit 9Z011 and the blocking bar kit 9Z010 or 9Z021 following the instructions for use of these kits.

Perform the following lining steps as usual (Parts 11 and 17 can be painted, nickel plated, ...).

Take care of the recommended tightening torques and adjust the mast and back positions (secure with Proteor glue ref. XC047 the screws that are not used for adjustments).

Do not reduce the mast section : breakage risk, guarantee loss.

Be sure that the extension limiting strap is adjusted so that the maximum extension of the mast end does not exceed 12 cm (to respect the original length), then secure with Proteor glue ref. XC047 the strap maintaining screw.

Check that the connection base / bearer (wheelchair or other) is strong enough to avoid any fall risk.

The C.P.O. is responsible for the compatibility of the accessories with each other and for their fastening.

Warning for the patient's family

Never use the seat-brace as a car seat.

Never carry the seat-brace with the patient inside.

Do not place the seat-brace near a flame or where the temperature is over 80° C (176 °F).

If a component is broken, do not use the seat-brace and do not try to repair it yourself.

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