1K10199-04/05

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1K101 INSTRUCTIONS FOR USE

**LIMITS OF USE**

- Use this product only as a socket alignment tool. It must never be assembled to a final prosthesis.
- No more than 10 uses.
- For patients up to 100 kg.

ASSEMBLY

Use the four milled head hex socket screws 7 to fix the 1K101 system to any "European Standard" component (four \varnothing 6 holes with centre distance 36 mm)

-e.g. 1K03, 1K30, 1K14, etc.

Use the four convex head hex socket screws 6 to fix the 1K101 system to any "European Standard" component (four M6 holes with centre distance 36 mm)

-e.g. 1K01, 1K02, 1K25, 1K90, etc.

ADJUSTMENTS

Translation is adjusted with screw 2. The range of adjustment is 15 mm (+ or - 7,5 mm).

Tilt is adjusted with screws 3. The range of adjustment is 18° (+ or - 9°).

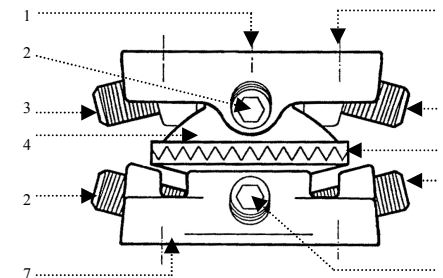
The direction of adjustment can be changed by rotating the shell casings 4 and 5 relative to each other. To do this, loosen screw 1. Then after rotating 4 and 5 as desired, retighten screw 1. The adjustment pitch is 8°30'

ADVISED TIGHTENING TORQUE VALUES

Screws 1 : 22 Nm

Screws 2 and 3 : 15 Nm

Screws 6 and 7 : 6 Nm

**CAUTION**

Never leave screw \uparrow or \downarrow in place if the head has penetrated more than 1 mm into the tapped hole. If this happens, replace the screw with one of the longer ones provided. Conversely, if a screw juts out too much, replace it with a shorter one provided.

Advice : after adjustment use on the threads of screws \uparrow and \downarrow average strength adhesive PROTEOR ref. XC047.

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